

VOLUME UP

Irresistible charm of volume





VOLUME UP

Treatment of wrinkles and folds, replacement of volume defects, smoothing and moisturizing the skin

PACKING


1 vial
x
1 box

EFFECT

- Skin texture improvement
- Skin regeneration
- Wrinkle reduction
- Long-lasting rejuvenation
- Moisturizing and hydrating the skin

INGREDIENTS

Rejuvenate

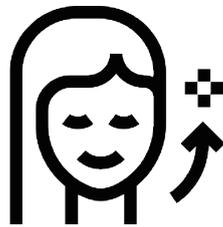

Poly-L-Lactic Acid
200mg

Moisturize



Na-Hyaluronic
Acid
40mg

Protect



Trehalose
10mg

RESULTS

2-3 weeks

- Skin texture improvement
- Skin feels more smooth

6-8 weeks

- Skin elasticity, firmness and density improves
- Skin looks healthier


3-6 months

- Fine lines and wrinkles smoothens
- More hydrated skin

12-24 months

- Skin remains youthful and firm
- Further care can prolong the effect

APPLICATION

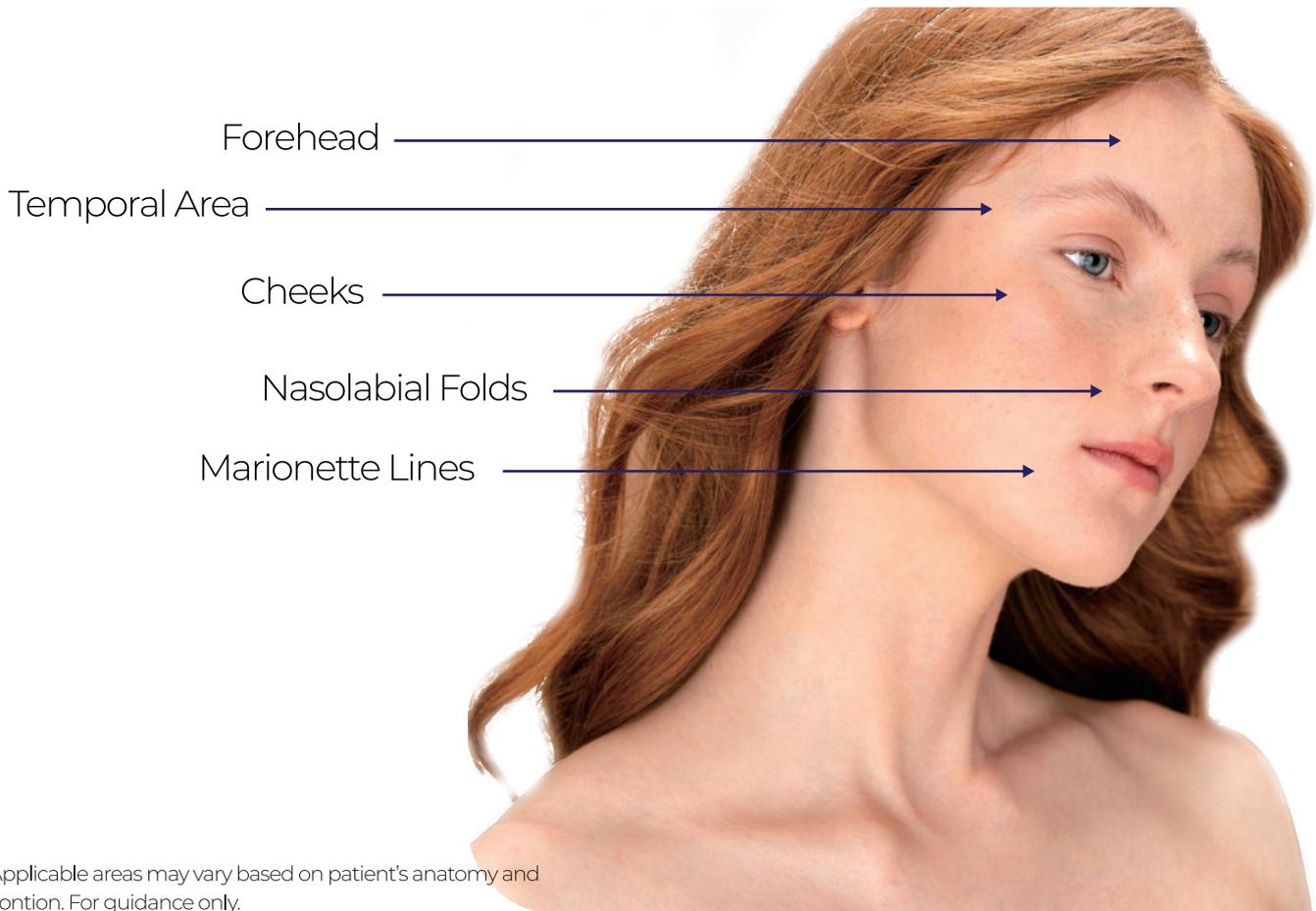

Applicable areas may vary based on patient's anatomy and condition. For guidance only.

PREPARATION

1. Make sure to check whether the product packaging is damaged or broken.
2. Check for any deformation of the product appearance or for any foreign materials attached to the product.
3. After removing the cap, make sure that the cap or rubber stopper is not damaged. In other case do not use the product.
4. Add sterilized saline water, 2% Lidocaine and shake the vial till everything mixed well.
5. Apply the product to the desired areas.

CAUTIONS

- Do not reuse the product. Once opened it should be used.
- Store at room temperature below 30°C, do not freeze.
- If the sealed package is broken, damaged or modified, do not use the product.
- Do not use if the patient has allergic reaction to the product or highly sensitive to the components.
- Do not use the product on inflammation or infected site

VOLUME UP

BEFORE & AFTER



1,5 months
after
treatment

3 months
after
treatment



#522 Dongkwang BizTower, 272 Sunhwagung-ro,
Namyangju-si, Gyeonggi-do, Republic of Korea
+82-2-6225-7988

Website: www.sbodyline.com

Email: dfkbio@dfkbio.com