


# SILKSENSE

High concentration HA Skinbooster with PN  
Lift & Rejuvenate with signature 7-point injection technique





## ★ Low molecular weight HA, High molecular weight HA

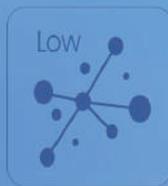

“ Low molecular weight HA retains moisture and hydrates the skin ”



“ High molecular weight HA gradually volumizes the skin in the dermis maintaining the lifting effect ”

## ★ Polynucleotide

Polynucleotide refers to a biocompatible substance derived from DNA that promotes tissue regeneration and stimulates collagen and elastin production.

## ★ What Makes It Unique ?

- Highly safe for the human body, as it does not contain cross-linking agents involved in chemical bonding
- By combining both low and high molecular weight hyaluronic acids, it hydrates the skin and provides long-lasting effects
- It enhances facial contours through collagen-boosting and gel-forming properties of PN
- HA, a key component of the extracellular matrix (ECM), helps create an optimal environment for skin regeneration

## ★ Product Information

Product Name	SILKSENSE
Packing unit	2.5mL/1syr
Storage/Shelf-life	1-30°C/24 months from date of manufacture
Main Ingredient	Non-cross linked HA 40mg/mL High molecular weight HA (1.7 Mda) 10mg/mL Low molecular weight HA (0.3 Mda) 30mg/mL Polynucleotide 2mg/mL
Purpose of Use	Temporarily improves moderate to severe nasolabial folds in adults by physically repairing the area through subdermal injection of hyaluronic acid and sodium polynucleotides

## How To Use

- Open the packaging of the product containing the syringe
- Gently remove the syringe stopper from the syringe
- Attach a 27- gauge, 13mm needle, which is the recommended size to the syringe prior to use
- Thoroughly disinfect the treatment area before injection
- Insert the needle into the treatment area and slowly inject an appropriate amount depending on the severity of wrinkles
- After treatment, massage the area to distribute the product gently

## Treatment Schedule

It is recommended to undergo 3-5 treatment sessions, spaced 2-4 weeks apart for optimal results

\* Please note that the intervals between treatments may vary depending on an individual's skin condition and specific needs

Results are typically observed 2-4 weeks post-treatment

This guide is intended to provide a general overview of injection techniques and administration considerations, ensuring optimal results while addressing the specific needs and sensitivities of each patient

## 7 Point Injection Guide



"The above treatment recommendations are general guidelines and may vary based on the patient's condition."

# SILKSENSE

DFKBIOLAB INC.

#522 Dongkwang BizTower, 272 Sunhwagung-ro, Namyangju-si, Gyeonggi-do, Republic of Korea (12106)

**T** +82 2 6225 7984 **E** [dfkbio@dfkbio.com](mailto:dfkbio@dfkbio.com) **W** [www.sbodyline.com](http://www.sbodyline.com)