



Excellent efficacy  
for skin  
around eyes

iREJU

# PN eye





## PACKING

1mL  
X  
0.6% PN  
+  
HA

## APPLICATION

- Reduces signs of skin aging around the eyes
- Smooths fine lines & wrinkles around the eyes
- Fixes dark circles and eyebags
- Improves naturally bright-eyed appearance

## INDICATION

Anti-aging treatment that activates revitalization of the skin, improves its rigidity, elasticity and tone.

## EFFECTS

- Helps with scar treatment
- Moisturizes skin tissues
- Enhances skin trophism
- Improves skin elasticity
- Minimizes pores



## MAIN INGREDIENTS

PN – Polynucleotides

– PN is a purer and more concentrated form of PDRN and has stronger regenerative effects

– PN is approved by the Ministry of Food and Drug Safety as a material safe for human use.

# HOW TO USE

---

## Under eye treatment

1. Apply to 6-10 spots under each eye
2. Distribute evenly 1 ml of solution per one eye (1ml for one session, one eye)
3. Repeat the procedure every 2~3 weeks three times (total of 3 procedures)



The treatment area, amount, method may vary depending on the patient's condition.

## RESULTS

---

### 3~5 days after

- Skin texture improvement
- Reduced sebum secretion

### 2~4 weeks after

- Increased skin elasticity
- Self-regenerating collagen production increase
- Pore size reduction

### 4~6 weeks after

- Natural lifting effect
- Skin volume increase

## POST TREATMENT

---

- Skin redness subsides within 1-2 hours.
- Makeup can be applied to skin 4 hours after the procedure.
- It is suggested to use hydrogel mask after the treatment, such as iREJU XO HYDROGEL MASK, to secure and prolong the effects
- Please use regenerating cream, moisturizing cream, and sunscreen to regenerate the skin after the procedure.
- Avoid using scrubs or peeling agents when washing your face.
- Avoid alcohol drinking, smoking, strenuous exercise, and saunas for one week after the treatment.
- After washing your face please refrain from using toner containing alcohol.

iREJU

# PNeye

1mL

POWERFUL SOLUTION FOR SKIN COMBINING PN + HA



#522 Dongkwang BizTower, 272 Sunhwagung-ro,  
Namyangju-si, Gyeonggi-do, Republic of Korea  
+82-2-6225-7988

Website: [www.sbodyline.com](http://www.sbodyline.com)

Email: [dfkbio@dfkbio.com](mailto:dfkbio@dfkbio.com)