



SARDENYATM Matrix

High concentration Dual-weight HA Skinbooster



What is **SARDENYA**TM **Matrix** ?

Unique skin booster formulated with pure hyaluronic acid that spreads naturally beneath the skin. It stimulates collagen and elastin production, enhancing skin firmness, hydration, and radiance from within.

“ Advanced formulation powered by Key Ingredients ”

01 Sodium Hyaluronate

Provides deep hydration, improves skin elasticity

02 Sodium Chloride

Helps maintain skin's natural balance, enhances absorption

03 Sodium Phosphate Buffer

Stabilizes pH for optimal skin compatibility

TECHNOLOGY

Premium Hyaluronic Acid Delivery System

Features a scientifically balanced formulation that combines 40mg of high molecular weight sodium hyaluronate, 40mg of low molecular weight sodium hyaluronate to deliver comprehensive hydration and skin rejuvenation effects.

The high molecular weight HA forms a protective layer on the skin's surface, creating a moisture-retaining barrier that prevents transepidermal water loss while providing immediate smoothing. In contrast, the low molecular weight HA penetrates deeper into the dermal layers to stimulate hydration from within, support collagen activity, and improve skin elasticity over time.

To ensure maximum skin compatibility and optimal efficacy, the formulation is stabilized with sodium chloride and a sodium phosphate buffer system, which maintain isotonicity and pH balance. These components help enhance absorption, minimize irritation, and preserve the integrity of the hyaluronic acid molecules throughout the treatment process.

Together, this dual-action HA complex provides both instant hydration and long-term skin firming benefits, making it ideal for restoring youthful skin texture, reducing signs of aging, and improving overall dermal health.


40mg of Hyaluronic Acid
High molecular weight

H-HA

MIX

L-HA

40mg of Hyaluronic Acid
Low molecular weight

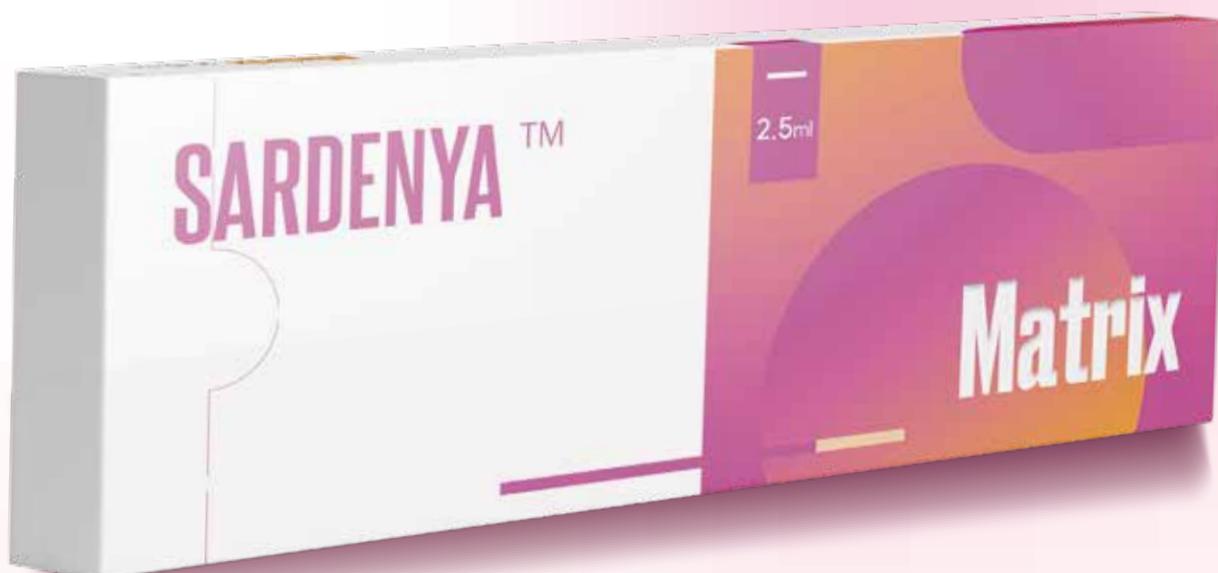

HOW TO USE

BAP (Bio-Aesthetic Points)

A standardized method using **5** injection points on each side of the face
Optimal diffusion and minimal bruising

These 5 points identify the 5 anatomically receptive areas of the face, avoiding large vessels and nerve branches. This minimizes risks while maximizing the diffusion of the product in the malar and sub-malar areas.

01

**Injection
Depth**

Deep dermis to superficial subcutaneous layer

02

**Dosage
per Session**

Total Dose: 2.5 mL per session
Injection Points: 5 points per side (total 10 points)
Volume Per Injection Point: 0.25 mL

03

**Injection
Timeline**

Initial Phase



Maintenance



"Treatment outcomes and duration may vary depending on individual patient conditions.
This information is intended for professional reference only."

01 Zygomatic protrusion

The highest point of the cheekbone
Supports mid-face volume and enhances cheek contour

03 Tragus

Anterior to the ear
Contributes to facial lifting effects

05 Chin

Below the mouth corner
Addresses volume loss in the lower face and enhances skin firmness

02 Nasal base

Lateral to the nose
Smooths fine lines around the nasolabial fold

04 Mandibular angle

Jawline angle
Helps reduce sagging and improves jawline definition

01

02

03

04

05



SARDENYATM Matrix

DFKBIOLAB INC.

#522 Dongkwang BizTower, 272 Sunhwagung-ro, Namyangju-si, Gyeonggi-do, Republic of Korea (12106)

T +82 2 6225 7988 E dfkbio@dfkbio.com W www.sbodyline.com